

Urine Specimen Collection Guide

For *Chlamydia trachomatis* and/or *Neisseria gonorrhoeae*

Collection for Male and Female Urine Specimens

Patient should not have urinated for at least 1 hour prior to specimen collection.



Maximum
volume 30 mL.

Direct patient to provide **first-catch urine** (approximately 20 to 30 mL of initial urine stream) into urine collection cup free of any preservatives. Collection of larger volumes of urine may result in specimen dilution that may reduce test sensitivity. **Female patients should not cleanse labial area prior to providing specimen.**

Specimen Transport and Storage

Urine samples in primary collection containers must be transported to lab at 2°C to 30°C **within 24 hours of collection.**



For Test Collection Supplies Please Contact:
Laboratory Alliance of Central New York, LLC
113 Innovation Lane, Liverpool, NY 13088
Ph: (315) 410-7067 Fax: (315) 410-7062
www.laboratoryalliance.com

APTIMA
Combo2

GEN-PROBE
Pioneering Advances in Healthcare
Through DNA Probes

10210 Genetic Center Drive
San Diego, CA 92121
(858) 410-8000
(800) 523-5001

www.gen-probe.com

© 2001 Gen-Probe Incorporated
L316-Rev A